Home Care Technical Specification

1 1/4" Grab Bars - Concealed Screw

8700

Stock Numbers:

- R8712W, DN8712, LR8712W 12" Grab Bar
 - Stainless Steel or White Finish, 1 1/4" Bar
- 8716, R8716W, LR8716W 16" Grab Bar Stainless Steel or White Finish, 1 ¼" Bar
- 8718, L8718, R8718PS, LR8718PS,
 R8718PB, LR8718W, R8718W 18" Grab
 Bar

Stainless Steel, Polished Stainless, Polished Brass, or White Finish, 1 1/4" Bar

 8724, L8724, R8724PS, LR8724PS, LR8724W R8724PB, R8724W – 24" Grab Bar

Stainless Steel, Polished Stainless, Polished Brass, or White Finish, 1 ¼" Bar

- 8730, R8730W 30" Grab Bar Stainless Steel or White, 1 ¼" Bar
- □ 8732, R8732W 32" Grab Bar Stainless Steel or White Finish, 1 ¼" Bar
- R8736PS, R8736W, 8736, L8736–
 36" Grab Bar
 Polished Stainless or White Finish, 1 ¼" Bar
- □ 8742, R8742PS, R8742W, R8748W– 42" Grab Bar

Satin or Polished Stainless Finish, White Finish, 1 1/4" Bar

□ 8748, R8748W – 48" Grab Bar Satin Finish or White Finish, 1 1/4" Bar

Materials:


1 ¼" Grab Bar rods are formed from type 304 stainless steel tubing.
12" – 42" length is 20 gauge
48" length is 18 gauge

Compliance:

- ADA, ANSI, ASTM, CSA
- Tested to 500 lbs. pull



Dimensions:

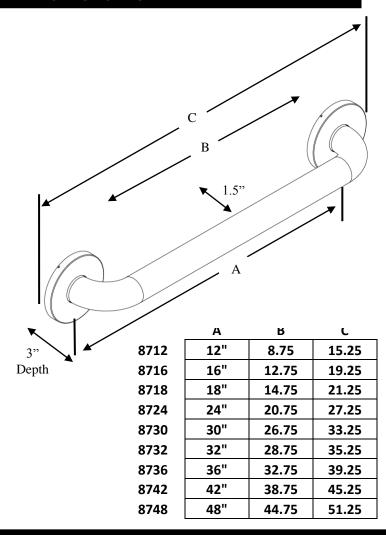

Installation Instructions:

- 1. Position unit on wall in desired location.
- 2. Mark mounting holes.
- 3. Drill starter holes.
- 4. Mount unit on wall using 2" screws (provided).
- 5. Snap flange covers into place.

Note: SecureMount Anchors are recommended for use in nonstud (hollow-wall) applications per installation instructions. (Secure Mount Anchors are sold separately)

Stock Number:	_1 ¼" Grab Bars
By Creative Specialties	